

# V-STEADY

Radiopaque bone cement for vertebral consolidation



# V-STEADY high viscosity bone cement


V-STEADY is an acrylic-based radiopaque bone cement with immediate development of viscosity and long working time.

Ready for use immediately after mixing (in less than two minutes), it maintains this consistency for approximately 9 minutes thanks to its specific formulation that "pauses" the polymerization reaction. V-STEADY bone cement is ideal to perform procedures on several vertebral levels and for who wants

a high-viscosity cement that maintains its properties throughout the useful working time. The cement completely hardens in 14 minutes.

It can be prepared by hand (bowl and spatula) or using closed mixing system PicoMix V. Because of its viscosity properties, it can be applied with high pressure gun (V-HP GUN) for controlled injection of high-viscosity cement.

- Reduced mixing time (under a minute to obtain a homogeneous product)
- Reduced waiting time
- Optimal dispensing control
- Working time: 9 minutes
- Excellent mechanical properties (especially the compression strength)
- High concentration of contrast medium (45% ZrO<sub>2</sub>)
- Low polymerization temperature so as to reduce the risk of thermal shock on the tissues

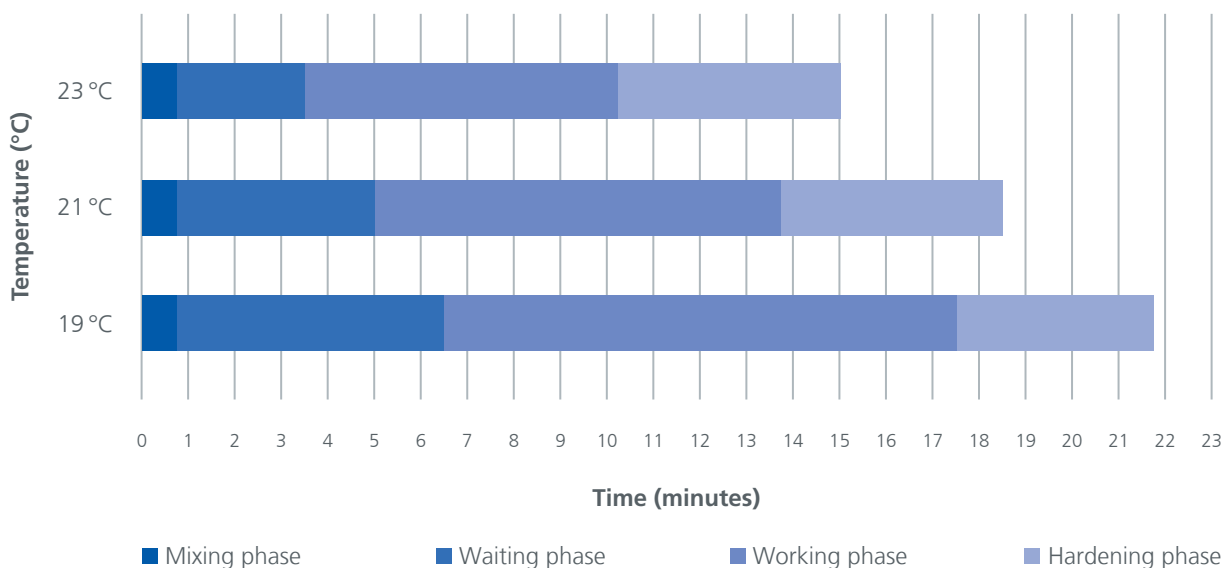



## Advantages

- Controlled injection with the V-HP Gun
- Its viscosity properties minimizes the risk of leakage during injection
- Up to 9 minutes of working time



V-HP GUN



V-STEADY and  
PicoMix V

# Components

Product	Description	REF
V-Steady	High viscosity radiopaque bone cement	800039
PicoMix V	Closed system to mix and dispense bone cement	900129
V-HP GUN	High pressure gun for radiopaque bone cement injection	900165
V-Access	Needle, bevel tip, 150 mm, Gauge 11	VV 11 150 5

# Clinical References

Kyphoplasty and vertebroplasty in the management of osteoporosis with subsequent vertebral compression fractures, G. Marcucci, M. L. Brandi, *Clinical Cases in Mineral and Bone Metabolism* 2010; 7(1): 51-60.

Vertebroplasty and Kyphoplasty for the Management of Osteoporotic Vertebral Compression Fractures, D. B. Pateder, A. J. Khanna, I. H. Lieberman, *Orthop Clin N Am* 38 (2007) 409-418.

Bone cements for percutaneous vertebroplasty and balloon kyphoplasty: Current status and future developments, Z. He, Q. Zhai, M. Hu, C. Cao, Ji. Wang, H. Yang, B. Li, *Journal of Orthopaedic Translation* (2015) 3, 1e11.

The clinical comparative study on high and low viscosity bone cement application in vertebroplasty, T.H. Zeng, Y. M. Wang, X. J. Yang, J. Y. Xiong, D. Q. Guo, *Int J Clin Exp Med* 2015;8(10):18855-18860.

A comparison of high viscosity bone cement and low viscosity bone cement vertebroplasty for severe osteoporotic vertebral compression

fractures, L. Zhang, J. Wang, X. Feng, Y. Tao, J. Yang, Y. Wang, S. Zhang, J. Cai, J. Huang, *Clinical Neurology and Neurosurgery* 129 (2015) 10-16.

An updated comparison of high- and low-viscosity cement vertebroplasty in the treatment of osteoporotic thoracolumbar vertebral compression fractures: A retrospective cohort study, Z. Zhang, J. Yang, H. Jiang, Z. Lai, *International Journal of Surgery* (2017).

Comparison of high- and low-viscosity cement in the treatment of vertebral compression fractures A systematic review and meta-analysis, Z. F. Zhang, H. Huang, S. Chen, D. Liu, MD, Y. Feng, C. Xie, F. Jiao, *Medicine* (2018) 97:12(e0184).

Bone cement, R. Vaishya, M. Chauhan, A. Vaish, *Journal of clinical orthopaedics and trauma*, 4 (2013), 157-163.

Mechanical performance of acrylic bone cements containing different radiopacifying agents, M. P. Ginebra, L. Albuixech, E. Fernandez-Barragan, C. Aparicio, F.J. Gil, J. San

Roman, B. Vazquez, J. A. Planell, *Biomaterials* 23 (2002) 1873-1882.

ISO 5833:2002 - Implants for surgery -- Acrylic resin cements.

Acrylic bone cement in total joint arthroplasty: A review., Saleh KJ; El Othmani MM; Tzeng TH; Mihalko WM; Chambers MC; Grupp TM, *J Orthop Res*; 2016 05; 34(5):737-44. PubMed ID: 26852143.

Spinal Compression Fracture Management: A Review of Current Treatment Strategies and Possible Future Avenues, I. K. Genev, M. K. Tobin, S. P. Zaidi, S. R. Khan, F. M. L. Amirouche, A. I. Mehta, *Global Spine J.* 2017 Feb;7(1):71-82.

Efficacy and Safety of High-Viscosity Bone Cement Vertebroplasty in Treatment of Osteoporotic Vertebral Compression Fractures with Intravertebral Cleft, Shenghui Tang, Wangjun Fu, Hongda Zhang, Haonan Zhang, Biru Liang.



G21 S.r.l. - Via S. Pertini, 8 - 41039 San Possidonio (MO) - ITALY - Tel. +39 0535 30312 - www.g-21.it - info@g-21.it

Over a Century  
of Innovation



Distributed by:  
**ulrich GmbH & Co. KG** | Buchbrunnweg 12 | 89081 Ulm | Germany  
Phone: +49 (0)731 9654-0 | Fax: +49 (0)731 9654-2705  
spine@ulrichmedical.com | www.ulrichmedical.com